




 NOTE



When using a technique for direct spot filming , I.I. indirect spot filming  or , DR, press the [X-ray switch]  on the R/F table console.

 Reference 6.6 "Spot Filming"

 NOTE

Make sure that the set exposure conditions are appropriate before performing radiography.
Performing radiography under inappropriate conditions may result in unnecessary exposure to the patient.

5.6.1 Radiography Using the Hand Switch

- 1** Press the hand switch to the first stage.
- 2** Confirm that the hand switch lights.
When radiography preparation is complete, the hand switch lights green and the  (radiography ready indicator) lights on the operation panel.
- 3** Press the hand switch to the second stage.
Radiography is conducted.
During radiography, the hand switch lights yellow and the  (radiography indicator) on the operation panel lights.
Hold down the hand switch until radiography is complete.

