

# RINCE L7

RINSE ADDITIVE AND NEUTRAL ACTIVE DRYING

SURGICAL INSTRUMENTS



AUTOMATIC WASHING MACHINE



## ■ PRODUCT BENEFITS

- Neutral pH
- Silicone free
- Rince additive for the LDI final rinsing cycle Additif to boost the drying process
- Rapid elimination of traces of moisture on instruments which are immediately dry and ready to be sterilised
- Bring a shiny appearance to instruments and improve articulation condition



**Fb Franklab®**  
notre expertise l'Ultra-Propreté



0.1%



# RINCE L7

## RINSE ADDITIVE AND NEUTRAL ACTIVE DRYING

### ■ INSTRUCTIONS FOR USE

#### ■ LDI final rinse cycle settings:

- Recommended concentration: .....0.03 to 0.1%
- Recommended temperature: .....90 to 93°C
- Recommended time contact : .....3 minutes

*Final rinsing cycle must be performed at 93°C during 3 minutes (A0=3000) to ensure a thermal disinfection.*

#### ■ Product to associate during the cycle:

- **Cleaning phase:**
  - TFD Premium (TDS 1730.B): ....inactivating prions detergent
  - ENZYMEX L7 (TDS 790.B): .....enzymatic detergent
  - TFD 7 (TDS 020.B): .....alkaline detergent
- **Intermediary rinsing:**
  - PHOSPHAX (TDS 050.B): .....acid neutralizer

### ■ PRECAUTIONS OF USE

See (SDS010329)

### ■ FEATURES

- Aspect: **Liquid**
- Color: **Yellow orange**
- pH at 0.1%: **7**
- Expiration date: **2 years**

### ■ COMPOSITION

- Non-ionic surfactants
- Rinse and dry activator agents
- Sequestrants

### ■ PACKAGING

- ① Box of 4 x 5L .....ref. **1032920**
- ② Box of 2 x10L .....ref. **1032922**  
Box of 3 low cans of 5L .....ref. **1032915 (on demand)**



Medical device Class 1.