	ÜRÜN ADI PRODUCT NAME	REFERANS NO REFERENCE NUMBER	ETİKET NO LABEL NO	VERSİYON VERSION	TARİH DATE
Medbar Tıbbi Malzemeler Turizm San. ve Tic. A. Ş. 1142 sok no:35 Sarnic- Gaziemir İzmir TURKEY 35414	Karmen Kanül Enjektör Kullanım Kılavuzu Karman Cannula Injector User Manual				

260,00 mm

**User Instructions**

- Begin with the valve button(s) open (not pressed), the plunger positioned all the way inside the cylinder, and the collar stop locked in place (with the tabs pushed down into the holes in the cylinder).
- Push the valve button(s) down and forward until you feel them locked.
- Create a vacuum by pulling the plunger arms snap outward and catch on the wide sides of the cylinder base. Both plunger arms must be fully extended to the sides and secured over the edges of the cylinder
- To test unlock the valve button(s) and feel the rush of air (indicator of pressure). If the rush of air is not strong, displace or remove the collar stop, withdraw the plunger and check that the plunger o-ring is free of damage and foreign bodies, properly lubricated and properly positioned in the groove. Also make sure the cylinder is firmly placed in the valve. Then create a vacuum and test it again. If vacuum is still not retained, discard and use another karman cannula syringe.
- Select and have the appropriate cannula size.
- Peel off the sterilization pouch of the cannula. Move it slowly into the uterine cavity until it touches the fundus, then withdraw it slightly
- Attach the cannula to the end of the syringe. For single valve syringe, it is not necessary to use any adaptor. For double valve syringe, use the appropriate adaptor as per the following. 4-5-6 mm (blue), 7mm(light brown), 8mm(white), 9mm (dark brown), 10mm(green), 12mm (no adaptor).
- Start suction by releasing the valve button(s). Suction will start immediately.
- Evacuate the contents of the uterus by rotating the cannula 180 degrees in each direction while using a gentle in-and-out motion.
- The instruments can be withdrawn when the uterus is empty.
- Empty the contents of the syringe into an appropriate container by releasing the button(s), squeezing the plunger arms and pushing the plunger fully in the cylinder. Dispose after the operation.

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