


2.5.5. To Replace the Lamp

 Follow this procedure to replace the lamp.

Access: **Main menu > Services > Customer Services > Cycles**

Consumables and spare parts: **Lamp bulb 1203601326**, **Light Bulb Dismantling Allen Key 1207821086**

1. Press **Replace Lamp**.
The lamp is switched off and the reagent and sample needles are positioned so as to make the lamp replacement easier.
A dialog box asks you to replace the lamp.
-



Warning! Hot surface

Risk of personal injury due to hot surfaces within the instrument. Wait for the lamp to cool down before proceeding.

2. Open the main cover.
3. Press the front of the optic hatch to open it.

