


Bio-group MEDICAL SYSTEM S.r.l. Loc. Campiano 9/b Talamello (RN) P.Iva 00964170419 Divisione Prove Interlaboratorio 	Istruzioni d'uso Instruction for use Urine e Sedimento Urinario Urine and Urine Sedimentation	I.O. 06 - IFU Rev. 01 Data 10/2020
--	--	--

REF: MSQUALITYU-MSQUALITYSD

CONTENUTO :
1 FLACONE - 10 ml

COMPOSIZIONE :
Un flacone pronto all'uso.

PRECAUZIONI D'USO
Prodotto ad uso professionale.
Leggere attentamente le istruzioni.
Conservare come indicato in metodica.

CONSERVAZIONE E STABILITA'
Stabile fino alla data di scadenza riportata sulla confezione
Il materiale nella confezione integra, se conservato a +2 / +8°C, è stabile fino alla data di scadenza riportata sull'etichetta. Non congelare.
Dopo apertura del flacone: 30 giorni a 2 - 25°C.

- MODALITA' D'USO**
- Portare il siero a temperatura ambiente almeno 1 ora prima di eseguire il test.
 - Agitare delicatamente con cura il campione per 1 – 2 minuti.
 - Controllare che il campione sia ben miscelato.
 - **NON CENTRIFUGARE**
 - Per sistemi automatici: trattare il campione come un normale campione di urina
 - Per analisi microscopica: depositare sul vetrino la quantità necessaria alla lettura
 - Inserire i risultati nel data entry sul sito qs-veq.it

N.B. ATTENZIONE. UTILIZZARE IL CAMPIONE CON LE NORMALI PROCEDURE DI LABORATORIO

REF: MSEQUALITYU-MSEQSUS

INLCUDES:
1 VIAL - 10 ml

COMPOSITION:
1 vial of Human sample stabilized ready to use

PRECAUTIONS:
Use this product under the professional technician's guidance.
Read the instruction carefully before use.
Store as required.

STORAGE AND STABILITY
Store the kit at 2-8 °C
Do Not Freeze

PROCEDURE

- Bring the sample to room temperature at least 1 hour before running the test
- Gently invert the vial for 1 to 2 minutes.
- Check whether the sample has been mixed well. If not repeat the previous step.
- **DO NOT CENTRIFUGE**
- For automated system: treat the sample as a routine human sample
- For microscope analysis: drop the sample on slide and proceed as a routine human sample
- Analyze the sample.
- Insert the results in the data entry

WARNING:
ATTENTION USE THE VIALS AS HUMAN SAMPLE

	Consultare istruzioni per l'uso / Read instructions for use		
	Scadenza / Expiry	REF	Numero di catalogo / Catalog Number
	Limiti temperatura di conservazione/ Storing temperature limits		Fabbricante / Producer
			Dimensione / Numero test / Dimension / Number tes
LOT	Numero di lotto / Lot Number		