




LIFECODES® PC-AHG
Anti-Human Globulin Positive Control Serum
Anti-Bw4, Bw6

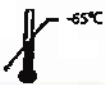
REF

405467

0.5 mL

RUO

For Research Use Only. Not for use in diagnostic procedures.

**INTENDED USE**

For use as a positive control reagent when performing AHG procedures.

SUMMARY/EXPLANATION

Anti-human globulin positive control serum is a blend of human sera that contains antibodies which react with Bw4 and Bw6. Each unit of sera used was tested and found to be non-reactive for HBsAg, Anti-HIV and Hepatitis C.

- Store frozen aliquots at $\leq -65^{\circ}\text{C}$.

WARNING: Human blood products have the potential to transmit disease. Take appropriate precautions.

Dispose of according to local regulations.

INSTRUMENTS/EQUIPMENT

$\leq -65^{\circ}\text{C}$ Freezer (Not included with reagent)

INSTRUCTIONS FOR USE

1. Thaw prior to use.
2. Aliquot vial to one time use volumes.
3. Store aliquots at $\leq -65^{\circ}\text{C}$.
4. Dispense 1 μL in micro-lymphocytotoxicity tray wells as needed.
5. Repeated thawing and freezing is detrimental to antibody reactivity.

REACTIVITY

Special Note: An initial titration of this antisera may be necessary to determine the dilution at which it should be used to obtain acceptable reactivity according to each laboratory's protocol.

1. This serum will result in a negative reaction when tested with lymphocytes in the standard NIH microlymphocytotoxic assay.
2. This serum will result in a negative reaction when tested with a wash microlymphocytotoxic assay.
3. This serum will result in a positive reaction when tested in an AHG + wash microlymphocytotoxic assay.

STORAGE/HANDLING/SHIPPING

This AHG positive control serum is shipped frozen, on dry ice. Store the frozen serum at $\leq -65^{\circ}\text{C}$ upon arrival.

REFERENCES

1. ASHI Laboratory Manual: Fourth Edition, 2000, Quality Control Program III.D, 1, page 3.



Immucor GTI Diagnostics, Inc.
20925 Crossroads Circle
Waukesha, WI 53186 USA

US and International Contact Information:

Technical Support : waukeshatechsupport@immucor.com

www.immucor.com

© 2002-2014 Immucor GTI Diagnostics, Inc.

405467.IFUEH Rev C
2015-01-12