



LIFECODES®

LIFECODES® PHS (PHS-N100, PHS-N5) Negative Control Pooled Human Serum Non Heat Inactivated

REFPHS-N100:
PHS-N5:405468
405510100 mL
0.5 mL**RUO**For Research Use Only. Not for use in
diagnostic procedures.

INTENDED USE

For use as a negative control reagent.

SUMMARY/EXPLANATION

Negative control serum is a pool of sera aseptically collected from whole blood. The whole blood is collected from non-transfused male donors of ABO type, AB. This material was tested and found to be non-reactive for Hepatitis B Surface Antigen (HBsAg), Anti-HIV and Hepatitis C.

- Store frozen aliquots at $\leq -65^{\circ}\text{C}$ indefinitely.

CAUTION

Human blood products have the potential to transmit disease. Take appropriate precautions.

Dispose of according to local regulations.

INSTRUMENTS/EQUIPMENT

$\leq -65^{\circ}\text{C}$ Freezer (Not included with reagent)

PROCEDURE

- Thaw prior to use. This product is not heat-inactivated.
- Thawed aliquots may be stored at 4°C for up to 24 hours.
- All lots of negative control sera should be pre-screened by the user laboratory to determine the suitability for the specific methods employed.

REACTIVITY

Determine negative values based on methods, targets and comparison to previous lot number of negative control.

STORAGE/HANDLING/SHIPPING

Pooled Human Serum is shipped frozen, on dry ice. Store the frozen serum at $\leq -65^{\circ}\text{C}$ upon arrival.

REFERENCES

- ASHI Governance, Standards for Histocompatibility Testing, 2002, American Society of Histocompatibility and Immunogenetics, Section E, Serologic Typing



Immucor GTI Diagnostics, Inc.
20925 Crossroads Circle
Waukesha, WI 53186 USA

US and International Contact Information:

Technical Support: waukesha技术支持@immucor.com

www.immucor.com

© 2002-2014 Immucor GTI Diagnostics, Inc.

405468.IFUE Rev D
2015-01-12
