


# Mission™ Urinalysis Reagent Strips Quick Reference Guide


Follow each step closely to ensure proper and accurate reading.

1. Remove the strip from the canister and close it tightly. Record the opening date on the canister label. Discard the strips **3 months** after the opening date.


2. Immerse the reagent areas of the strip in urine for **1-2 seconds** then **immediately remove** the strip from urine.

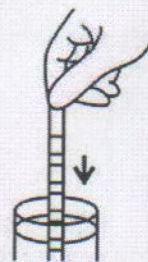

3. Run the **edge** of the strip against the **edge** of the urine container to remove excess urine.

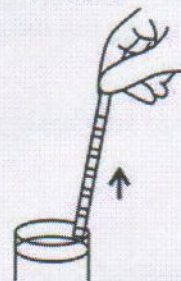


4. Hold the strip in a horizontal position and **touch the edge of the strip on absorbent paper** to avoid mixing chemicals from one pad to another and soiling hands with urine.

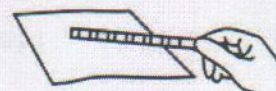

5. Read results visually or by analyzer reading.

Note: For visual reading, hold the strip close to the color blocks and match carefully at specified times. Results may be read up to 2 minutes after the specified times.

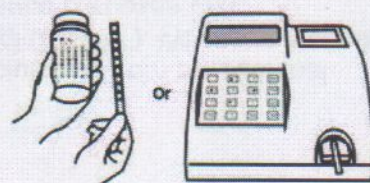

## Notes:

- The negative color blocks on the color chart label **do not** correspond to the reagent areas of the dry strip. The negative color blocks on the color chart label **should only be compared after** immersing the strips in urine.
- Refer to package insert or instruction manual for complete information.

**ACON**

ACON Laboratories, Inc. 4108 Sorrento Valley Boulevard, San Diego, CA 92121, U.S.A.

