



# NEUTRALIZATION SOLUTION FOR AUTOMATIC INSTRUMENT WASHER MACHINES

## DETRO MATIC NS – USER MANUAL

---

- **Neutralization solution**
- **Concentrate**

### **PRODUCT PROPERTIES**

Detro Matic NS is used for removing alkaline remnants after alkaline cleaning of instruments at washer machines.

Removes alkaline residues.  
Provides Effective Cleaning  
Suitable for use in all washing machine brands.  
Compatible for All Water Hardness  
Protection Against Corrosion  
Neutralizer activation at temperatures  $\leq 45^{\circ}\text{C}$   
Low Usage Concentration Powerful Effect

### **INGREDIENT**

>20% Citric Acid (Cas no: 77-92- 9), Complex Formers, Corrosion Inhibitors.

In order for the product to wash effectively at the correct dosage, the device manufacturer's recommendations must be followed. For better results, use with Detro Matic A (alkaline solution).

### **DOSING**

It is used at %0,05-0,1 concentration. (5-10 ml product into per 10 lt water)

### **WARNING**

Read usage instruction before usage. Don't use for cleaning of hand, face and food. Keep out of reach children. Irritant agent. Don't contact with skin and mucous membranes. In the case of eye contact, wash plenty of water immediately.

### **STORAGE CONDITIONS**

Keep away from direct sunlight. Store containers tightly closed and in a well ventilated area at 0-30 °C

### **PACKAGE**

5 L

**DETRO HEALTHCARE KİMYA SANAYİ A.Ş.**

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr