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Five Continents Medical Instrument

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NINGBO FIVE CONTINENTS MEDICAL INSTRUMENT CO., LTD.

五洲医械
Five Continents Medical Instrument

COMPANY INTRODUCTION

The First Manufacturer in China in Disposable Surgical Lavage System for effective debridement following orthopedic procedure of trauma & joint arthroplasty ,Ningbo Five Continents Medical Instrument Co., Ltd. was installed by few top orthopedic surgeons in Sep. 2004 in Yuyao, Zhejiang province,China.All our products are certified by CE & ISO13485.

Our Surgical Lavage System combines convenience and power for the removal of contaminated water,necrotic tissue,bone debris,wound pollutants and foreign material whilst providing controlled pressure for efficient cleansing and irrigation on the site of flat wound surface or osteotomy surface or the intra-medullary.

Products are keeping improving via carrying out clinical researches & feedback from surgeons,noises & weight(only 772g now) are minimized,which are favorable by surgeons.It is one essential medical instrument in modern orthopedic procedures.

We are in striving to develop & try our best to promote updated-products for the maximal sucess of each surgery and the minimal pain of each patient.

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Disposable Surgical Lavage System

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Surgical lavage system is fully disposable, it is designed for orthopedic arthroplasty bone-bed preparation, with strong cleaning and removing bone & cement debris & strong suction function for residual fluid during total joint procedure and trauma cases.

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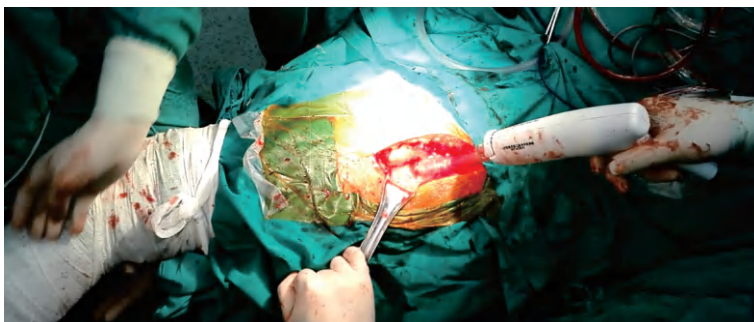
五洲医械
Five Continents Medical Instrument

Application/应用


In total knee arthroplasty procedure /全膝关节置换


In total hip arthroplasty procedure /全髋关节置换

In spinal surgery /脊柱手术

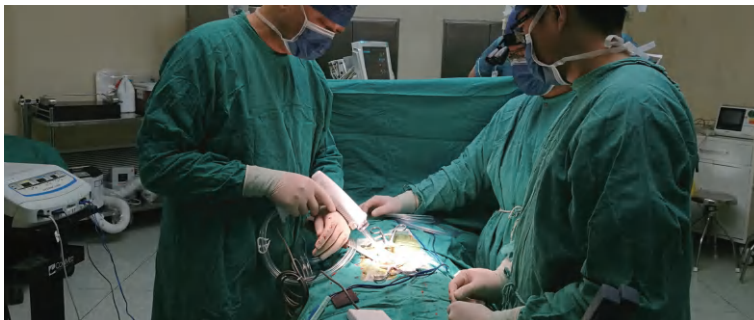In trauma case /创伤、清创

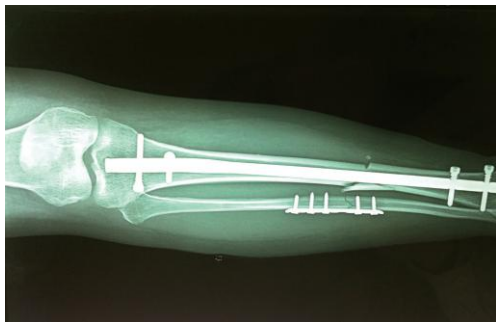五洲医械
Five Continents Medical Instrument

Advantage/优势

Clinical Advantage

- Increase penetration between bone cement and osteotomy surface.
- Reduce the incidence rate of fat embolism.
- Reduce the loosen rate of the implant, reduce reconstruction rate for artificial joint arthroplasty.
- Effective and rapidly cleansing of necrotic tissues, bacteria and foreign materials, minimizes risk of wound infection, accelerate wound healing.
- Enlarge operation vision, shorten operation time.
- Reduce the use of the dosage of antibiotics, save cost.
- Decrease the post-operative rate of adhesion.

Applicable Department:

- Orthopedic department: orthopedics, orthopedic tumour, orthopedic spine and orthopedic trauma.
- Department of thoracic & abdominal surgery: thoracic department, general surgery, obstetrics & gynaecology, urology surgery, microsurgery, trauma emergency, burn department and other debridement in burn & skin graft, diabetic and pressure ulcers, chronic and infected wound.

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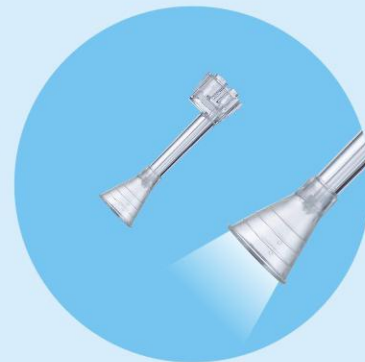
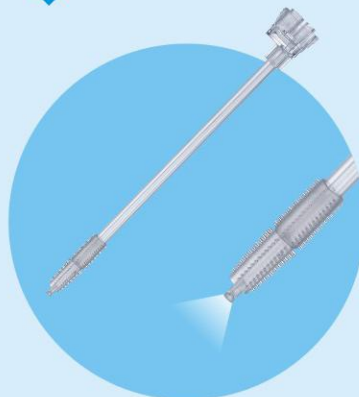
髓腔内冲洗 带刷长喷头
Brush Tip For intra-medullary Debridement


Short Tip For Flat Surface Debridement
创面冲洗 短喷头

The brush tip with radial spray function is available for your selection which can be purchased separately.

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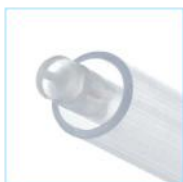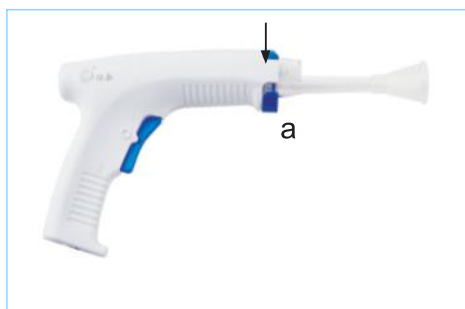
Press up the locking ring(a),choose the needed tip(the short one is for site irrigation,the long one is for cleansing of deep tissues or the bone marrow cavity).

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Split two tubes,insert the puncturer into the saline (or irrigation fluid) bag,connect suction tube onto the vacuum extractor(provided by hospital).

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Two speeds are optional,fast or slow speed according to procedure requirement.Switch the power trigger,cleans the wound.Please turn off the flow-control switch if the cleansing is paused.

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National Patent
for application

NINGBO FIVE CONTINENTS MEDICAL INSTRUMENT CO., LTD.
宁波五洲医疗器械有限公司

Add.: No. 63-9 Modao Bridge Road, Yuyao city, Zhejiang province, China.

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