# Iris System Cleanser

**REF** 800-3203

## TABLE OF CONTENTS

|                                      |    |
|--------------------------------------|----|
| English . . . . .                    | 2  |
| Français (FR) . . . . .              | 5  |
| Deutsch (DE) . . . . .               | 8  |
| Italiano (IT) . . . . .              | 11 |
| Español (ES) . . . . .               | 14 |
| Português Portugal (PT-PT) . . . . . | 17 |
| Dansk (DA) . . . . .                 | 20 |
| Svenska (SV) . . . . .               | 23 |
| Norsk (NO) . . . . .                 | 26 |
| Ελληνικά (EL) . . . . .              | 29 |
| 日本語 (JP) . . . . .                   | 32 |
| 中文 (ZH-CN) . . . . .                 | 35 |
| Lietuviškai (LT) . . . . .           | 37 |
| Magyar (HU) . . . . .                | 40 |
| Polski (PL) . . . . .                | 43 |
| Čeština (CZ) . . . . .               | 46 |
| Slovenčina (SK) . . . . .            | 49 |
| 한국어 (KO) . . . . .                   | 52 |
| Türkçe (TR) . . . . .                | 55 |
| Русский (RU) . . . . .               | 58 |
| eesti keel (ET) . . . . .            | 61 |
| Hrvatski (HR) . . . . .              | 64 |
| Български (BG) . . . . .             | 67 |
| 中文 (ZH-TW) . . . . .                 | 70 |
| Română (RO) . . . . .                | 72 |
| Србија (SR) . . . . .                | 75 |
| Latviešu (LV) . . . . .              | 78 |
| Українська (UK) . . . . .            | 81 |
| Português Brasil (PT-BR) . . . . .   | 84 |
| Tiếng Việt (VI) . . . . .            | 87 |

# Iris System Cleanser

**REF** 800-3203

**Rx Only in the U.S.A.**

## INTENDED USE

Iris System Cleanser is intended for *in vitro* diagnostic use with iQ200 or DxU 840/850m Series of Automated Urine Microscopy Analyzers and iChemVELOCITY or DxU 810c Automated Urine Chemistry Analyzer.

## INTENDED USER

The device is intended to be used by a laboratory professional.

## PRINCIPLE

Iris System Cleanser is a ready-for-use solution specifically developed as a wash solution for the iQ200 or DxU 840/850m Series of Automated Urine Microscopy Analyzers. For the iChemVELOCITY and DxU 810c Automated Urine Chemistry Analyzers, the Iris System Cleanser is used as a cleaning solution. A wash cycle must be performed at the beginning of each work day to prevent accumulation of residue in the fluidic system, sample lines and flow cell.

## WARNING AND PRECAUTIONS

- Use only Iris System Cleanser for the Wash Cycle. Other similar solutions of hypochlorite should not be used because of the risk of particulate contaminants, trace oils or releasing compounds that could cause blockage or damage to the optical window.
- This material is harmful to eyes, skin and if inhaled or ingested. Refer to the Safety Data Sheet for first aid measures.
- Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly.
- Do not reuse dispensed System Cleanser.
- Wear appropriate personal protective equipment when handling.
- Systemic allergic reactions are possible for sensitive individuals.
- Consult local or state authorities with regard to recommended method of disposal.
- Do not mix with other products- can release dangerous gas (chlorine).
- Product should present as a pale yellow, clear liquid; bleach odor. Product that differs from these attributes should be discarded.

## GHS HAZARD CLASSIFICATION

Iris System Cleanser

**DANGER**

H314

Causes severe skin burns and eye damage.

H400

Very toxic to aquatic life.

H411

Toxic to aquatic life with long lasting effects.

P273

Avoid release to the environment.

P280

Wear protective gloves, protective clothing and eye/face protection.

P301+P330+P331

IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

P303+P361+P353

IF ON SKIN (or hair): Rinse skin with water.

P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310

Immediately call a POISON CENTER or doctor/physician.  
Sodium Hydroxide 0.1 - 1%  
Sodium Hypochlorite 1 - 5%

Safety Data Sheet is available at [beckmancoulter.com/techdocs](https://beckmancoulter.com/techdocs)

## REACTIVE INGREDIENTS

Sodium hypochlorite, 4%

## NON-REACTIVE INGREDIENTS

Purified Solvent

## STORAGE AND STABILITY

- Store at 20°C to 28°C (68°F to 82.4°F).
- Iris System Cleanser is stable for eighteen months from date of manufacture.
- Open expiration: Unused solution that remains in the original capped container is stable until the expiration date.
- Do not freeze.
- Do not store in direct sunlight.
- Do not use solution after the expiration date printed on the bottle label.

## MATERIALS PROVIDED

- 4 X 425 mL Iris System Cleanser
- 4 Flip-Top Caps

## MATERIALS REQUIRED, BUT NOT PROVIDED

- Approved tubes listed in the iQ200 or DxU 840/850m Instructions for Use and the iChemVELOCITY or DxU 810c Instructions for Use. Do not use preservative tubes.
- Personal Protective Equipment.

## PROCEDURE

- Follow the directions in the instrument IFU (Instructions for Use).

## TECHNICAL SUPPORT

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-854-3633 (USA or Canada) or contact your local Beckman Coulter Representative.

A Glossary of Symbols (PN C27648) is available at [beckmancoulter.com/techdocs](https://beckmancoulter.com/techdocs).

## NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

## TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see [www.beckmancoulter.com/patents](https://www.beckmancoulter.com/patents).

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Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100