

ENZYMATIC CLEANING SOLUTION FOR AUTOMATIC INSTRUMENT WASHER MACHINES, MANUAL AND ULTRASONIC

DETRO MATIC ENZYM- USER MANUAL

- ENZYMATIC CLEANING PRODUCT
- CONCENTRATE

Product Properties

Compatible For All Washer Machines

Compatible For All Water Hardness

Cleaning and Disinfection of Heat-Resistant Instruments at 93 °C

Cleaning of Thermolabile Instruments at 50-60 °C

Protection Against Corrosion

Low Usage Concentration

Powerful Effect

Usage Areas

It is used for machine enzymatic cleaning of medical instruments made from metal, glass, porcelain, plastic and other equipments.

Ingredient

Enzymes (amylase, protease, lipase), amphoteric tensides, complexing agents, corrosion inhibitors

Dosing

At machine, it is used at % 0,2 - 0,8 (20 - 80 ml product to 10 lt water) concentration, Manually it is used % 1 (10 ml product to 10 lt water) concentration.

Warning

Read usage intruction before usage. Don't use for cleaning of hand, face and food. Keep out of reach children. Irritant agent. Don't contact with skin and mucous membranes. In the case of eye contact, wash plenty of water immediately.

Storage Conditions

Keep away from direct sunlight. Store containers tightly closed and in a well ventilated area at 0-30 °C

Package

5 L

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr