CERTIFIED COMPANY UNI EN ISO 9001 & UNI CEI EN ISO 13485

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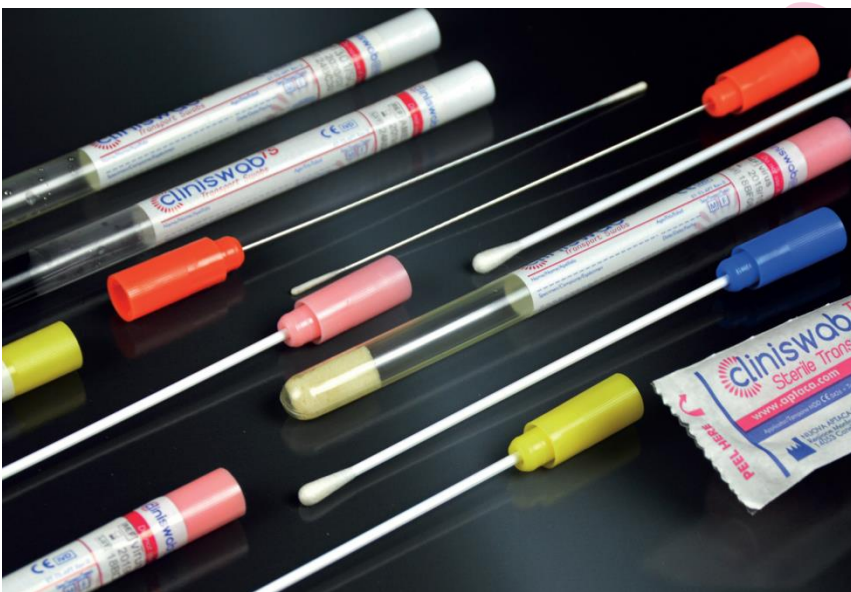
DESCRIZIONE / DESCRIPTION

A complete range of swabs for the collection and transport of bacteriological samples, with transport medium agar gel to grant the survival of specimens during their transportation to clinical laboratory. The transport

swabs are composed of a soft rayon tip with plastic or aluminum applicator stick, atoxic and resistant to sudden changes of temperature, and a labelled cylindrical test tube Ø12x155 mm, in unbreakable medical PP, to preserve sample and users safety. supplied with label on test tube and cap for patient and sample identification and as a seal indicating that the product has not been previously used. Cap has same diameter than test tube.

All the Devices are produced according to GMP (Good Manufacturing Practice) and are classified and certified as Medical Devices

Class IIa sterile (invasive surgery for temporary use). The swabs are sterile irradiated (SAL 10⁻⁶), in medical easy opening peel-pack, Pyrogen free.

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CE 0426 MD marked sterile Swab - manufactured in compliance with 93/42/EC Directive (Italian D.lgs 46 del 24/02/1997) + 2007/47/EC - Class IIa sterile

CE IVD marked sterile Test Tube with medium - manufactured in compliance with 98/79/EC Directive (Italian D.lgs 332 del 08/09/2000)

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Swab Intended purpose is "MEDICAL DEVICE" CLASS IIA STERILE. Swabs are sterile and ready-for-use systems intended for clinical samples drawing, transport and maintenance for cultural exam. The swab is suitable for short contact with the human body including surgical wounds. Test Tubes are classified as IVD and are intended only for the preservation of samples. Product must be used as directed, Aptaca is not responsible for any unauthorized use.

For professional use only.

National classification of medical devices (CND - For Italian law) > V901302 (Transport swabs for collection sample)

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APPLICATOR / TAMPONE

The applicator shaft of the swab is mounted on the tube plug which serves as an optimal grip for an efficient swab handling.

Available in two versions on sampling site base:

with rayon tip, atoxic. Particularly suitable for mouth, throat, vagina, urogenital apparatus, skin, wounds and surgical wounds.

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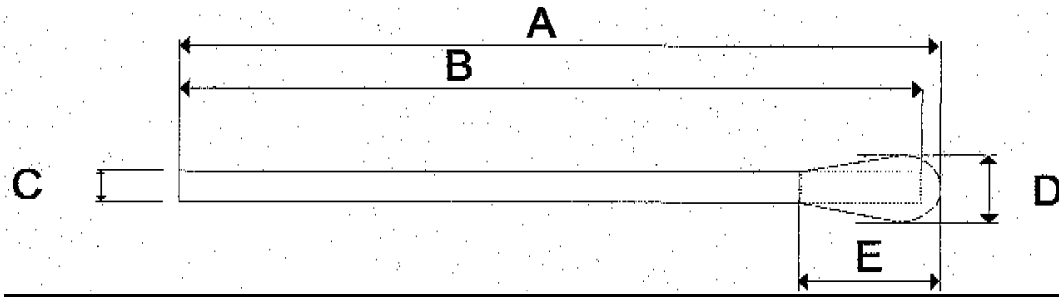
PLASTIC STICK, RAYON TIP / ASTA PLASTICA, PUNTALE IN VISCOSA

This specialty swab is made with rayon fiber. It is bonded securely to a rigid white birch polystyrene handle by an aqueous based adhesive.

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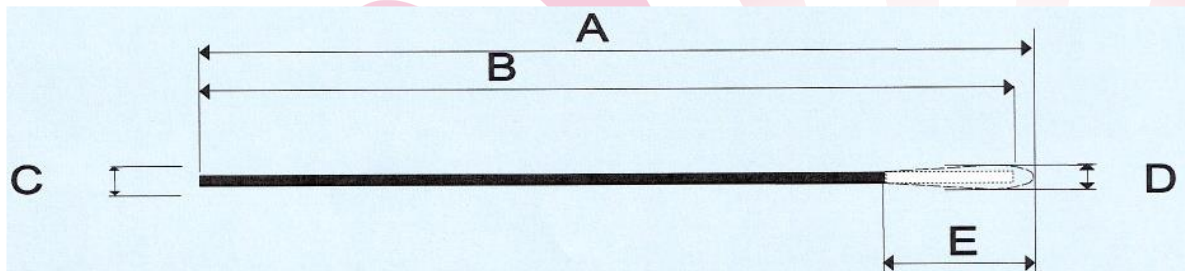
Absorbance / Assorbenza: 100 μ L \pm 10% in 10 sec. (acqua + TSB) / (water + TSB)

Dimensions / Dimensioni

A: 150 \pm 2 mm / **B:** 146 \pm 1 mm / **C:** \varnothing 2,5 \pm 0,1 mm / **D:** 5 - 6 mm / **E:** 15 mm \pm 2 mm

ALUMINUM STICK, RAYON TIP / ASTA ALLUMINIO, PUNTALE IN VISCOSA

Swabs with viscose (rayon) tip, aluminium stick, atoxic, resistant to sudden changes of temperature. Atoxic glue to fix viscose tip. Very thin shaft and small tip for paediatric, urethral, urogenital and nasopharyngeal sampling. Without break point.

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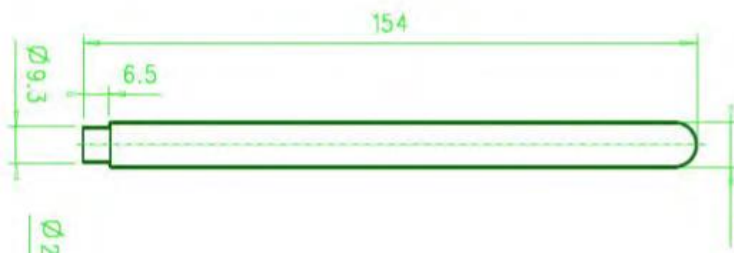
Absorbance / Assorbenza: 20 μ L \pm 10% in 10 sec. (acqua + TSB) / (water + TSB)

Dimensions / Dimensioni

A: 147 \pm 2 mm / **B:** 145 \pm 1 mm / **C:** \varnothing 0,9 \pm 0,1 mm / **D:** 2,2 \pm 0,5 mm / **E:** 12 \pm 2 mm

TEST TUBE / PROVETTA

Transport Medium is contained in a cylindrical test tube \varnothing 12 x 155 mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and user's safety. With pressure cap in polyethylene (HDPE) with specific colour according to medium type. Pre-labelled on test tube to allow a correct sample identification. Item code, lot number and expiry date printed on each label for a full traceability.

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AMIES with CHARCOAL / AMIES con CARBONE

Sterile swabs with Amies with charcoal agar gel medium. The Amies formula with charcoal has been shown to improve the recovery of *Neisseria gonorrhoeae*, *Vibrio cholerae* and is often moreover recommended for *Bordetella pertussis*.

Available with double Amies agar gel medium (6 ml) which allows a better bacteriological sample inoculation inside the medium, ensuring a safer anaerobic bacteria viability. Furthermore it allows to dilute the samples in a medium greater quantity.

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STUART with CHARCOAL / STUART con CARBONE

Sterile swabs with Stuart with charcoal agar gel medium. The Stuart formula with charcoal has been shown to improve the recovery of *Neisseria gonorrhoeae*.

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CARY BLAIR

Sterile swabs with Cary Blair agar gel medium. Particularly suitable for rectal swabs and faecal specimens. It is suitable for the transport of Salmonella, Shigella, Campylobacter and Vibrio. Sodium glycerophosphate presents in Stuart medium, is replaced by buffer phosphate to avoid, in the medium, the growth of bacteria that have glycerophosphate reductase. The medium is particularly suitable for the transport of Salmonella and Shigella: furthermore its high pH relatively alkaline minimize the bacterial destruction due to formation of acid. Viability of Vibrio cultures for up to four weeks.

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Use of the Cliniswab^{TS} in conjunction with rapid diagnostic kits and instruments must be validated prior to use by the user.

Bacteria survival in the transport media depends upon a series of factors such as bacteria type, transport time, preservation temperature, bacteria concentration in the sample and transport medium formula.

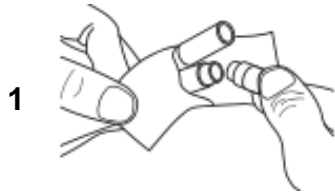
*Devices ensure the survival of several micro-organisms for 24-48 hours. In the case of fastidious bacteria such as *Neisseria Gonorrhoeae* and *Streptococcus pneumoniae*, the samples on the swab shall directly be put on a plate with culture medium or immediately transported to the laboratory and cultivated within 24 hours.*

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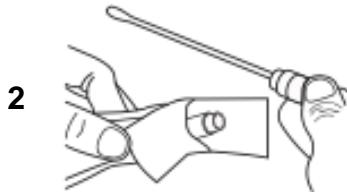
All raw materials, components and every lot of finished product undergo strict quality inspections carried out by N.A. in order to check conformance to specifications.

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The content of the unopened and undamaged units is guaranteed to be sterile. Do not use the units in case of damage, dehydration or contamination. The Devices must be stored at +2°C (35.6°F) / +30°C (86°F) until used. Improper storage will result in a loss of efficacy. Do not use after expiration date indicated on label.

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Cliniswab® is a registered trademark of Aptaca S.p.A. All rights reserved

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UNI EN ISO 9001-2015 (ISO 9001-2015)

per i seguenti Processi
concerning the following kinds of Processes

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Management of the manufacturing and placing on the market of sterile tampons for sampling of biological specimens in natural orifice and in surgical field. Design and manufacturing of diagnostic medical devices for laboratories of analysis. Management of the manufacturing and placing on the market of invasive medical devices with respect to body orifices (class I sterile). Manufacturing of invasive medical devices with respect to body orifices (class I sterile). Marketing of medical and diagnostic devices in vitro. Marketing of laboratory articles.

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CE Notification Confirmation

This is to confirm that, according to the council directive 98/79/EC, SUNGO Europe B.V. performed the notification duties and responsibilities as the European authorized representative of:

Shandong Chengwu Medical Products Factory
Southern End of Quancheng Road, Chengwu County,
274200 Heze City, Shandong Province, P.R.China.

The Manufacturer has provided SUNGO Europe B.V. with the EC Declaration of Conformity confirming that the In Vitro Diagnostic Medical Device (IVDD), as stipulated here below, is fulfilling the applicable requirements of the European Council Directive 98/79/EC.

According to 98/79/EC, the European Databank on Medical Devices (EUDAMED) is established as of May 1 2011. The Netherlands Competent Authority is notified of the manufacturer's In Vitro Diagnostic Medical Device and has allocated registration number.

Disposable Virus Specimen Collection Tube

Not in List A and List B according to Annex II of 98/79/EC

GMDN CODE : 63232

CIBG Number: NL-CA002-2020-50479

Where the manufacturer affix's the CE marking to the product listed they must ensure that all the requirements of the appropriate EU directive(s) have and continue to be met.

This document should be used together with the competence authority notification letter and the Declaration of Conformity issued by the Manufacturer. This document will become to be invalid once the notification status is changed or the EAR agreement is terminated.

Reference Number: EUCAN00195

Issue date: May, 06, 2020

SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE
Amsterdam, Netherlands
ec.rep@sungogroup.com

For and on behalf of
SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Authorized Signature
Only used for the EU Representative Signature

REGISTRATION NO. 04720Q10000336

CERTIFICATE OF QUALITY MANAGEMENT SYSTEM FOR MEDICAL DEVICES

This is to certify that the quality management system of

Shandong Chengwu Medical Products Factory

Registered Address: Southern End of Quancheng Road, Chengwu County, 274200 Heze City, Shandong Province, P.R. China

Manufacturing Address: Southern End of Quancheng Road, Chengwu County

Has been assessed and conformed to the following standard(s)

YY/T 0287-2017 idt ISO 13485:2016

The certificate is valid for the following scope:

The development, production and service of disposable virus specimen collection tube.

Date of issue: July 13, 2020

Date of expiry: July 12, 2023

General Manager:

**BEIJING HUA GUANG CERTIFICATION
OF MEDICAL DEVICES CO., LTD.**

Note: This certificate will not be continuously valid until the organization has been approved in the annual surveillance audit. The certificate information are available on the website of the certification and accreditation administration of the People's Republic of China (<http://www.cnca.gov.cn>) or the website of CMD (<http://www.cmdc.com.cn>). Address: 5th floor of Zhong Lian building, No. jia88, An Ding Men Wai street, Dongcheng district, Beijing, 100011, P.R. China Telephone: 010-62351993

EU Declaration of Conformity

Regarding In Vitro Diagnostic Directive (98/79/EC)

Manufacturer: SHANDONG CHENGWU MEDICAL PRODUCTS FACTORY
Southern end of Huxin road, Chengwu county, Heze city, Shandong province, P. R. CHINA.
Fax: +86-530-8623064

European Representative: Sungo Europe B.V.
Olympisch Stadion 24, 1076DE Amsterdam, Netherlands
ec.rep@sungogroup.com

Product Name: Swab

GMDN CODE: 57940

Classification: General, (IVDD, Article9(1)) NOT BE PART OF LIST A & B OF ANNEX II

Conformity Assessment Route: ANNEX III (EXCEPT POINT 6) OF IVDD 98/79/EC

We herewith declare that the above mentioned product meets the transposition into national law, the provisions of the following EC Council Directives and Standards. All supporting documentations are retained under the promise of the manufacturer.

We, as manufacturer, are exclusively responsible for this Doc.

Standards Applied:

EN ISO 15223-1:2016 EN 1041:2008+A1:2003 EN ISO 13485:2016
EN ISO 14971:2012 EN ISO 18113-1:2011

Place, Date of Issue: Chengwu, Shandong, 2020-03-01

Signature: 

Name: 

Position: General Manager

SHANDONG CHENGWU MEDICAL PRODUCTS FACTORY

Product Name: Disposable Virus Specimen Collection Tube

Shelf Life: 12 months

Store Under: Room Temperature 4-25℃

Product Description:

Virus collection kit is used for collection, transport, maintenance and long term freeze storage of Viruses, such as COVID-19 Virus, Influenza, including H1N1, Chlamydia, Mycoplasma and Ureaplasma specimens. The Virus sampling tube swab kit can be conveniently stored at room temperature.

Sampling tube:

The tube body and tube cap are Sterilized by Gamma Radiation , Non heat source, DNase and RNase free. To ensure that the sampling tube itself will not have a toxic effect on the sample. The tube body and cap are made of high-quality medical grade polypropylene material, which is transparent and clean, without deformation at high temperature and high pressure (120 ℃ 15min), and does not become brittle at low temperature (-80 ℃)

Sample Requirements

The collected flock swab samples should be transported at 2°C-8°C and submitted for inspection immediately. Sample transport and storage time should be no later than 72h.

User Manual

1. Mark the sample information on the label before sampling.
2. Collect the specimen with the swab.
 - a. Nasal swab: gently insert the swab into the nasal cavity for sampling, use another swab to collect another nasal sample in the same way.
 - b. Throat swab: collect the throat sample.
3. Put the swab into the transport tube contains virus transport medium after sampling.
4. Snap the swab at the break point into the transport tube and screw the cap tightly.
5. It is better to transport the collected samples to the laboratory with ice packs.

Pay Attention

1. When collecting and handling specimens, gloves, protective clothing, goggles and other personal protective equipment shall be worn to avoid splashing, leakage and exposure of potential pathogens.
2. All biohazardous specimens and devices shall be disposed of in accordance with the relevant regulations and procedures and discarded in the designated collection containers.
4. After use, the waste shall be disposed according to the regulations of the hospital or the environmental protection department.
5. Read Instructions for Use on this packing carefully before use, and use it within the validity period.
6. Warning and prevention: it is forbidden to use when package damage, foreign matters, leakage, storage liquid turbidity and expiration date are found.

Kit include: 1pc 10ml sampling tube with 2ML VTM LIQUID or 3ML VTM LIQUID.

- 1pc Nasal flocced swab
- 1pc Oral flocced swab
- Sterile blister pack.

| Cat No. | Description | Qty/Case |
|---------|---|----------|
| 611903 | Virus collection kit with 2 swab,(1pc nasal swab+1pc oral swab) | 300 |

Manufacturer

Shandong Chengwu Medical Products Factory

Add:Southern End of Huxin Road, Chengwu County, Heze City, Shandong Province P.R. China