


## 2.6.2 X-ray tube alignment

Tabletop tilting angle for X-ray tube alignment is zero degree. Align the center of image and X-ray tube alignment tool.

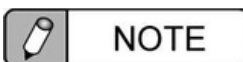

If changing SID from 1100 to 1500 with the tabletop tilting angle of zero degree, adjust that the marker of X-ray tube alignment tool does not misalign from the center of image.

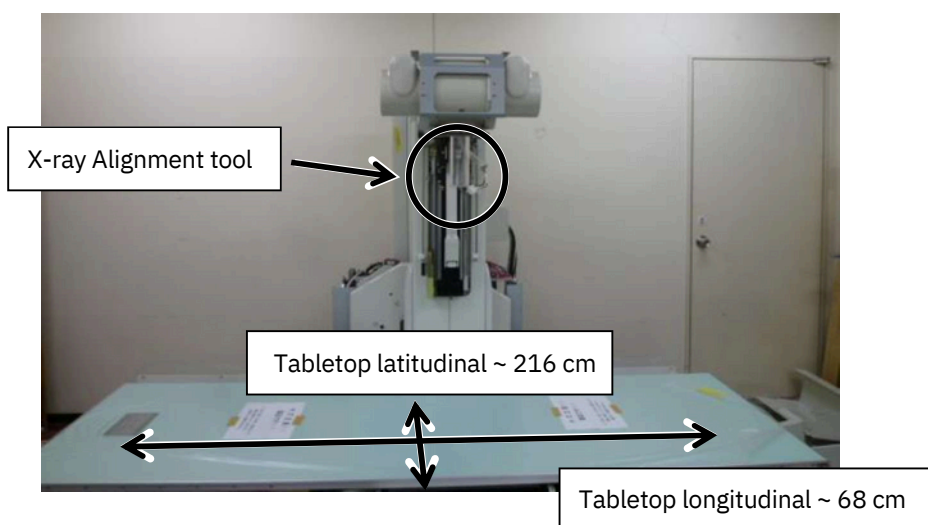


Fig. 2.6.2 X-ray Alignment



Refert to 15 [Appendix 3] Showing the cross cursor on the ACQ-PC monitor" in DR-300 Installation Manual for how to display the cross cursor on the display.



Fig. 2.6.3 X-ray Alignment (Monitor)